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Thank you for choosing the new automatic Amplicord 95S electrolarynx, a simple and functional device to fit your lifestyle.

Contents of the Box [A]

- | | |
|--|------------------|
| ❶ Amplicord 95S | ❷ Security Strap |
| ❷ Power charger AC 220V 50Hz
DC 9V 30mA | ❸ Slip Case |
| ❸ 8.4V 250mAh rechargeable battery | ❹ Dial stopper |
| | ❺ User's Guide |

Product Description [B]

- | | |
|----------------|-----------------|
| ❶ Device Head | ❷ Volume Dial V |
| ❷ Power switch | ❸ Battery cover |
| ❸ Tone Dial T | ❹ Strap Hook |

Getting Started with your Device [C]

1. Unscrew the battery cover.
2. Insert the batter, matching the + and - contacts.
3. If desired, attach the strap.
4. If desired, insert the dial stopper.
5. Screw the battery cover back on.

Recharging the Battery [D]

1. Switch off the device by sliding the switch towards the symbol **O**. If you use the dial stopper, you may switch off the device by rotating the stopper.
2. Unscrew the battery cover.
3. Remove the battery.
4. Insert the battery in the battery charger and plug it into a power outlet. It will take 8/10 hours to fully recharge the battery.
5. Remove the charged battery and insert it into the electrolarynx, aligning the contacts.
6. Screw the battery cover back on.

Instructions for Use [E]

1. Switch on the device by sliding the power switch towards the symbol **I**.
2. Apply the device onto the softer part of the neck, with a light and even pressure. The device will automatically begin functioning upon contact with the skin.
3. Test the device by vocalising the letters **A - E - I - O - U**.
4. *If you have inserted the dial stopper, slide it down to access the dials.*
5. Use the **T** dial to adjust to the desired tone.
6. Use the **V** dial to adjust to the desired volume.
7. *If you have inserted the dial stopper, slide it back up to lock the dials at the desired settings.*

Maintenance and user rules

Maintenance will be carried out by the dealer.

We recommend cleaning the device on a weekly basis using a cloth with a little amount of denatured alcohol. Do not use any other type of solvents.

Keep away from heat sources and do not immerse in water or other liquids.

In case of problems due to malfunctions, defects, unexpected events or situations and for the maintenance procedures of the device, please contact the manufacturer or the local distributor.

The device is for single person use.

You must report any serious incident that occurs in relation to the medical device supplied by us to the manufacturer and to the competent authority of the Member State in which you are located.

Environmental conditions of transport and storage

The product does not require particular transport and storage conditions.

The battery has a duration of approximately 1 hour and 30 minutes of continuous use and has a life cycle of approximately 1 and a half years.

The life expectancy of the device is estimated to be around 10 years.

Troubleshooting

Problem	Cause	Solution
The device does not vibrate.	The switch is off.	Move the switch to the on position.
After moving the switch, the device still does not vibrate.	The battery may be low or dead.	Recharge or replace the battery.

Warranty

The Amplicord 95S carries a 2-Year Limited Warranty (24 months) for each of its components, valid from the Date of Purchase. The device is guaranteed against manufacturing defects (defined as a defect in materials or workmanship). The warranty states that a free replacement and/or repair will be provided for any faulty components, during the applicable warranty period. This warranty will lose validity should the device show signs of manumission, or if the defect is a result of improper use or accidental drops or falls. The power charger and the rechargeable battery are excluded from this warranty.

Label symbols

Manufacturer	
Refer to instruction manual	
Serial number	
Unique device identifier	
CE Marking	
Medical device	
Catalogue number	
Date of manufacture	
Type CF applied part	

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