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Thank you for choosing the Amplicord D1 digital electrolarynx, a mix of innovation and design to best serve your communication needs.

Contents of the Box [A]

- | | |
|---------------------------|----------------|
| ❶ Amplicord D1 | ❺ Slip Case |
| ❷ Power charger | ❻ User's Guide |
| ❸ 9V rechargeable battery | |
| ❹ Security Strap | |

Product Description [B]

- | | |
|----------------|-----------------------------|
| ❶ Device Head | ❺ Power charger port |
| ❷ LED | ❻ Strap hook |
| ❸ Joystick | ❼ Battery Compartment Cover |
| ❹ Power switch | ❽ Strap hook |

Getting Started with your Device [C]

1. Slide the battery cover to remove it, by applying a light pressure on the three notches.
2. Insert the battery, matching it to the + and - symbols on the bottom of the battery compartment.
3. Attach the strap.
4. Close the battery compartment, making sure it is properly shut.

Recharging the Battery [D]

1. Make sure that the device is switched off, by checking that the power switch is in the top position.
2. Connect one end of the charger to a wall plug, and the other end to the power charger port at the bottom of your device.
3. The LED will turn red: the battery is now charging.
4. It should take 8/10 hours to fully charge the battery.

Instructions for Use [E]

1. Turn on the device by sliding the switch to the lower position. The device will vibrate lightly and the LED will blink green every 2 seconds. Should the battery levels be low, the LED will blink green 8 times, in rapid succession.
2. Apply the device onto the softer part of the neck, with a light and even pressure. The device will automatically begin functioning upon contact with the skin.
3. Test the device by vocalising the letter **A - E - I - O - U**.
4. Adjust the tone of the voice created by the device, moving the joystick to the left to decrease it or to the right to increase it.
5. Adjust the volume of the voice created by the device, moving the joystick downwards to decrease it or upwards to increase it.
6. **Once you have reached the desired tone and volume, wait for 15 seconds without moving the joystick in order to let the device automatically save your preferred setting. As the settings are saved, the device will vibrate and beep once.**

Maintenance and user rules

Maintenance will be carried out by the dealer.

We recommend cleaning the device on a weekly basis using a cloth with a little amount of denatured alcohol. Do not use any other type of solvents.

Keep away from heat sources and do not immerse in water or other liquids.

In case of problems due to malfunctions, defects, unexpected events or situations and for the maintenance procedures of the device, please contact the manufacturer or the local distributor.

The device is for single person use.

Do not use the device when it is charging.

You must report any serious incident that occurs in relation to the medical device supplied by us to the manufacturer and to the competent authority of the Member State in which you are located.

Environmental conditions of transport and storage

The product does not require particular transport and storage conditions.

The battery has a duration of approximately 1 hour and 30 minutes of continuous use and has a life cycle of approximately 1 and a half years.

The life expectancy of the device is estimated to be around 10 years.

Troubleshooting

| Problem | Cause | Solution |
|---|---------------------------------|-------------------------------------|
| The device does not vibrate. | The switch is off. | Move the switch to the on position. |
| After moving the switch, the device still does not vibrate. | The battery may be low or dead. | Recharge or replace the battery. |

Warranty

The Amplicord D1 carries a 2-Year Limited Warranty (24 months) for each of its components, valid from the Date of Purchase. The device is guaranteed against manufacturing defects (defined as a defect in materials or workmanship). The warranty states that a free replacement and/or repair will be provided for any faulty components, during the applicable warranty period. This warranty will lose validity should the device show signs of manumission, or if the defect is a result of improper use or accidental drops or falls. The power charger and the rechargeable battery are excluded from this warranty.

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